

INSTRUCTIONS FOR USE

Always use COTTONSEED OIL as a lubricant.

Apply cottonseed oil liberally to the internal and external uterine body prior to each use. Spread oil by hand to coat all surfaces inside and out.

Attempting to move the uterus/rectum without lubrication can result in damage such as punctures and tears.

Prolpase should be stored inside barrel to prevent damage or unnecessary strain to the labia.

A handle is located on the medial aspect of the uterus. The use of this handle is how you can displace the uterus. Lubricate first with cottonseed oil (inside and out), reach through the vulva, grab handle and pull uterus out. *Do not use anything other than hands to invert or replace the uterus.

A teaching coach or facilitator can provide realistic positive pressure against the uterus/rectum by pushing to displace the uterus while the learner attempts to correct the prolapse.

ASSEMBLY


Assembled as shown below. 1/4" by 1 1/2" full thread bolts, washers and nuts

Cottonseed oil holder can be installed anywhere on the barrel you would prefer. You can also install inside the barrel using the same mounting bolts.

Full Ring is mounted on the most external aspect of the labia.

*** IMPORTANT

Fasteners should be tightened to fully compress the silicone to the point that the tension will be distributed around the circumference of the tissue and not only at the bolt holes. Tension dependence and weight bearing at the point of the bolt holes could result in damage to the tissue.

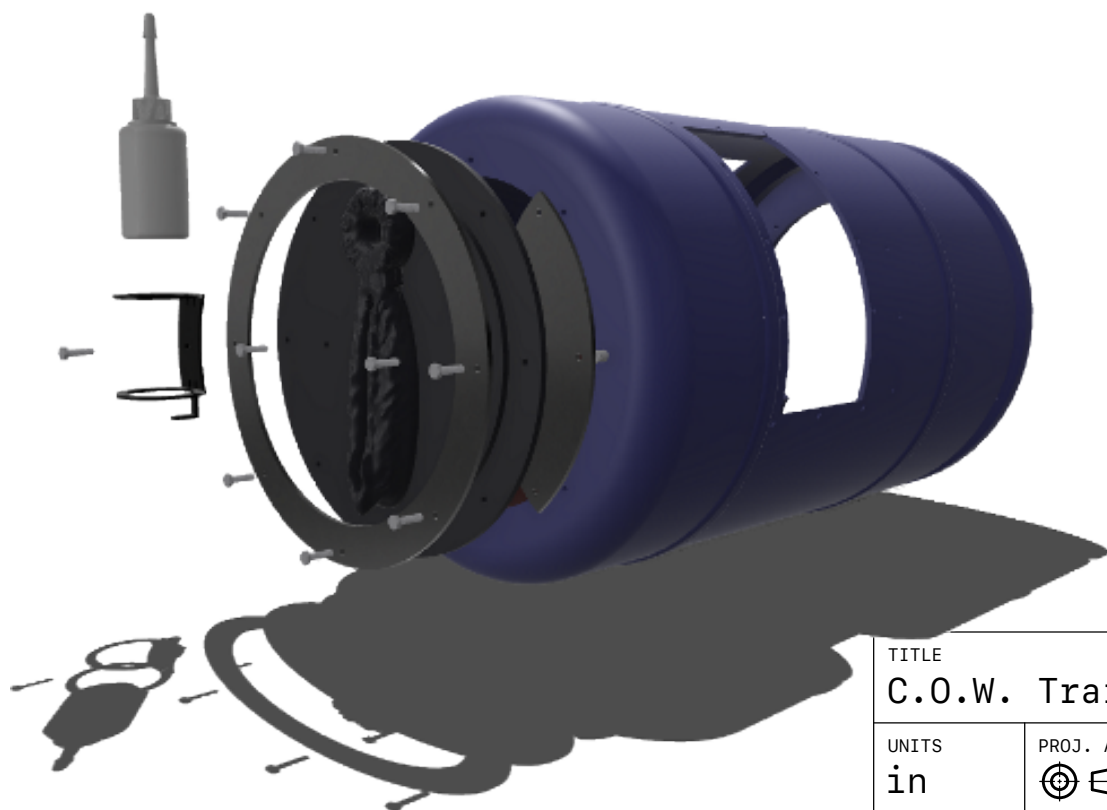

TITLE

C.O.W. Training System

UNITS

in

PROJ. ANG.



SIZE

A

SCALE

1:8

LAST UPDATED

08/01/26

SHEET

1/1